

TAKE-HOME WHITENING WITH OPALESCENCE GO® WHITENING



COSMETIC PREFILLED TRAYS
6% Hydrogen Peroxide
with Potassium Nitrate and Fluoride



Opalescence Go® whitening features ready-to-go convenience with an enhanced tray design for an even more adaptable and more comfortable whitening experience. After just 10 minutes in the mouth, the UltraFit™ tray softens and molds to the arch to create a custom-like fit that is remarkably comfortable and effective. And its molar-to-molar coverage ensures the gel comes in contact with more posterior teeth. Opalescence Go® whitening is available in 6% H₂O₂

(60-90 minutes wear time) and delicious Mint and Melon flavors. Opalescence Go tooth whitening gels contain PF (potassium nitrate and fluoride) to strengthen enamel and reduce sensitivity. Twenty percent water content prevents dehydration and shade relapse. After the first application in the dental office, the whitening trays can conveniently be worn at home.

Instructions



Center tray on arch.



Gently suck down or swallow.



Remove outer tray.



Wear 6% for 60 to 90 minutes.

Clinical Applications



Before whitening.



After 6 days of treatment with Opalescence Go whitening for 90 minutes.



UltraFit tray before.



UltraFit tray after 10 minutes in the mouth.